

Instructions For Use

Level 3 Surgical Mask with Ear Loops

Product Description

The Level 3 Surgical Mask is a single-use face mask for use in health-care where a person is at risk of blood or fluid splash.

Designed to meet Australian Standard AS 4381:2015, the mask limits the passage of blood or body fluids through the mask layers.

Featuring multiple layers of lightweight material, the mask provides comfort and breathability with two ear loops to hold the mask in place. The Level 3 Surgical Masks is available in two ear loop lengths and head tie option.



Applying the Mask

Appropriate hand hygiene should always be used when touching the mask. Masks should not be reapplied and should be discarded safely after use.



1 Hold the mask on the short edges with the ear loop connection points facing outward.



2 Pinching firmly on the nose piece, place one ear loop around the back of your ear and ensure it is secure.



3 Place the second ear loop up over your other ear and check the mask is evenly aligned on your face.



4 Expand the pleats to ensure a secure and comfortable fit.



5 Check mask fit to ensure it is secured at the bridge of the nose and mouth. There should be minimal gaps between the mask and face.



Appropriate hand hygiene should always be used when touching the mask. Masks should not be reapplied and should be discarded safely after use.

Removing the Mask

Prior to removing the mask, ensure hand hygiene is undertaken.

1. Without touching the mask, follow ear loops and unhook from the ears keeping tension on the bands.
2. Lean forward maintaining control of the ear loop tension, whilst slowly removing the mask away from the face. Ensure the mask does not flick forward or back onto the face.
3. Discard safely. Ensure hand hygiene is performed once the mask has been discarded.



Caution: The mask is designed for occupational use by healthcare professionals who are properly trained in mask use and limitations. Misuse may result in sickness or death. DO NOT use in any manner not indicated in the Instructions for Use. Always read the label. Discard the mask if it becomes damp, soiled, damaged, or difficult to breathe through. Do not use if the packaging is damaged.



Detmold Medical
45 Chief St, Brompton
South Australia 5007
Detmoldmedical.com

